

How To Calibrate Vet Tech 2030

Step 1: Turn off the display unit

Step 2: Turn on the display unit and immediately press hold and lbs/kg together at the same time briefly

The display should read “cal”

Step 3: Use lbs/kg button to select lbs or kg

Step 4: Press hold

Step 5: F'S – 6

Step 6: Press hold

Step 7: DP – 1

Step 8: Press hold

Step 9: DP- 1

Step 10: Press hold – the display should read “CAP”

Step 11: Press hold

300.0 change the digits that are flashing with lb/kg button, move to the next digit to the left with the zero button

Step 12: Press hold – the display should read “load”

Load is the weight you will use to calibrate with

Step 13: If the flashing digit is not correct change it with the lbs/kg button and use the zero button to move to the next digit

Step 14: Once the correct weight is displayed press hold – the display should read “cal”

Step 15: Press hold – display should read “no load”

Which means the weight should not be on the scale

Step 16: Press hold – the display should read “add load”

Step 17: Put the weight back on the scale and press hold

Step 18: Take weight off the scale and you are finished